


NeoPulse™

Lightweight, Adjustable and Disposable Pulse Oximeter Wrap

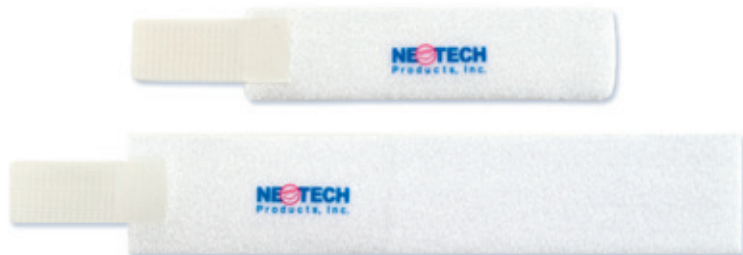

NeoPulse gently wrapped around foot.
Pulse oximeter not included.

FEATURES & BENEFITS

- Made with unique, patented NeoFoam®
- Soft and ultra skin-friendly
- Easily adjustable to fit any size
- Lightweight and comfortable
- Provides stable, secure attachment that won't slip
- Breathable material resists development of foot odor
- Wicks away perspiration

Cat No.	Size	Dimensions	Qty/Unit
N781	Small	3/4" x 3"	100/box
N782	Large	1" x 5"	100/box



Patent Pending **Made in USA**

1-800-966-0500 | www.neotechproducts.com

27822 Fremont Court, Valencia, CA 91355

NeoPulse™ Directions For Use

Step 1

- Apply pulse oximeter sensor per hospital protocol.

Step 2

- Wrap NeoPulse around the pulse oximeter sensor and secure with Velcro®. May be cut to fit.



Step 3

- Check the foot after applying to ensure adequate circulation, taking care to not wrap too tightly.

Step 4

- Change the wrap every 5-7 days or per hospital protocol.