



Introducing the Neotech RAM Cannula®
A Revolutionary Nasal Cannula

As gentle as a kiss

New

Neotech RAM Cannula®

Developed by a Neonatologist for Babies and Clinicians

Neotech introduces a new nasal cannula. Neotech Products, Inc.—a worldwide leader in the development of unique neonatal respiratory products, is pleased to introduce the Neotech RAM Cannula.



FEATURES & BENEFITS

- Soft, curved prongs
- Kink-resistant tubing
- Simple for clinicians to manage, comfortable for babies
- Kangaroo Care is easy
- Latex, Phthalate/DEHP, BPA free



This cross section shows the unique design that prevents tube from kinking.

Color-coded O-ring for comfort and easy identification of sizes.

Modular connector allows flexibility for various oxygen tubing sizes.

"We all love the simple application of the RAM and now it doesn't hamper positioning the baby. Kangaroo care is also a snap with the RAM."

Respiratory Therapist

See back panel for order information.

Patents Pending

New

Neotech RAM Cannula®

Developed by a Neonatologist for Babies and Clinicians



DIRECTIONS FOR USE

1. Select proper size cannula according to size chart.
2. Open package and remove cannula.
3. Attach cannula to oxygen source. Remove oxygen tubing adapter if needed.
4. Secure cannula to face with Neotech EZ-Hold cannula holders or skin friendly tape.
5. Change cannula every 30 days or per hospital protocol. When using heated and humidified air, change cannula every 14 days or per hospital protocol.



Oxygen administration from delivery to discharge.

Cat No.	Size	Approx. Patient Range	Qty/Unit
N4091NS	Preemie	<1000 grams	10/box
N4092NS	Newborn	1000 -2500 grams	10/box
N4093NS	Infant	>2500 grams	10/box
N4820	Oxygen Adapter	-	50/box

*To be used on or by order of a physician